


Vitrési Methods File and Firmware


Upgrade Instructions

To update the Vitrési Methods File and/or Firmware follow the instructions below:


1. Download the latest Vitrési Methods File/Firmware from the Helena Biosciences Europe website link onto a USB drive (<https://www.helena-biosciences.com/vitresi-downloads>).
2. Ensure the files are placed onto the top level of the USB and not within a sub folder.
3. Insert the USB drive into the USB port on the left side of the SAS Vitrési.
4. On the home screen, select **Maintenance**.

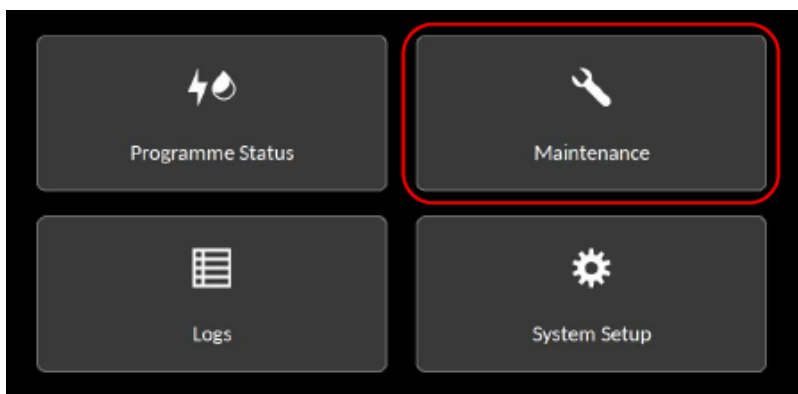

5. Within the **Maintenance** section, select **Firmware Update** (Note: this should be selected when updating either the Methods File or Firmware)
6. Any firmware or method files detected from the USB will appear on screen.

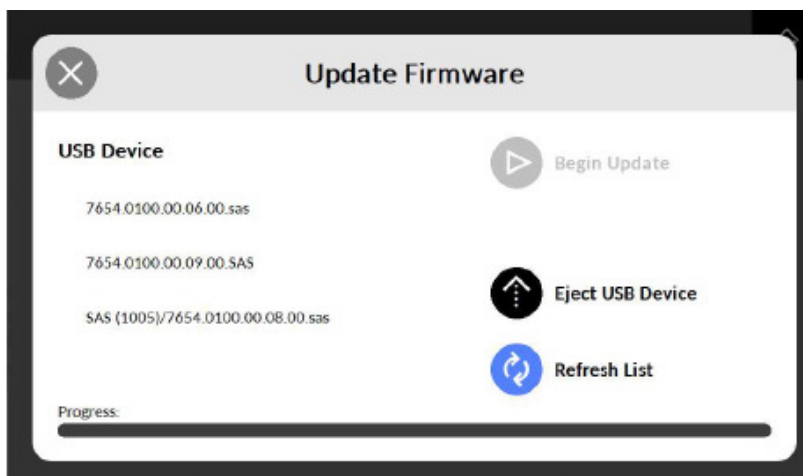


7. Select the appropriate file to update (this will be highlighted in blue).

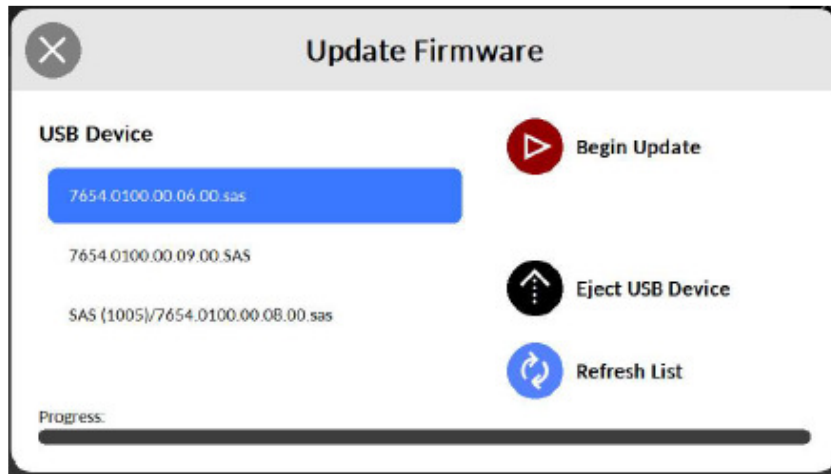


8. Press **Begin Update** to begin the update process. The update progress is displayed in the progress bar at the bottom of the screen.


When the update is complete the progress bar will be filled.

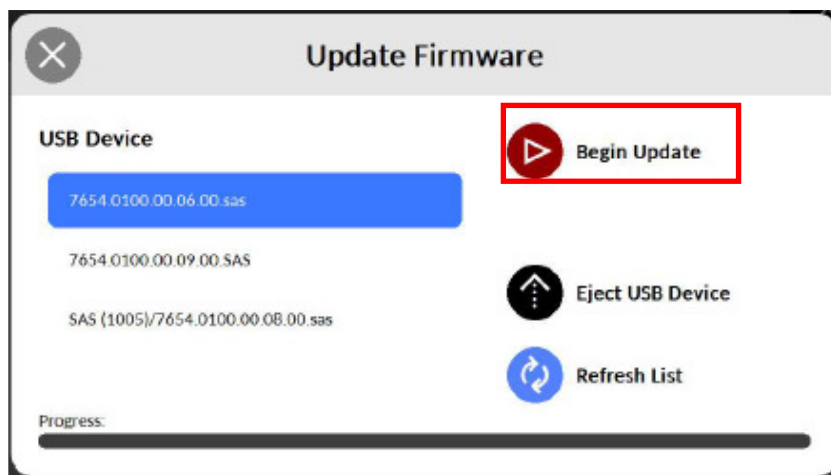

9. Once completed, remove the USB stick and select **Reboot** to restart the SAS Vitrési Instrument.

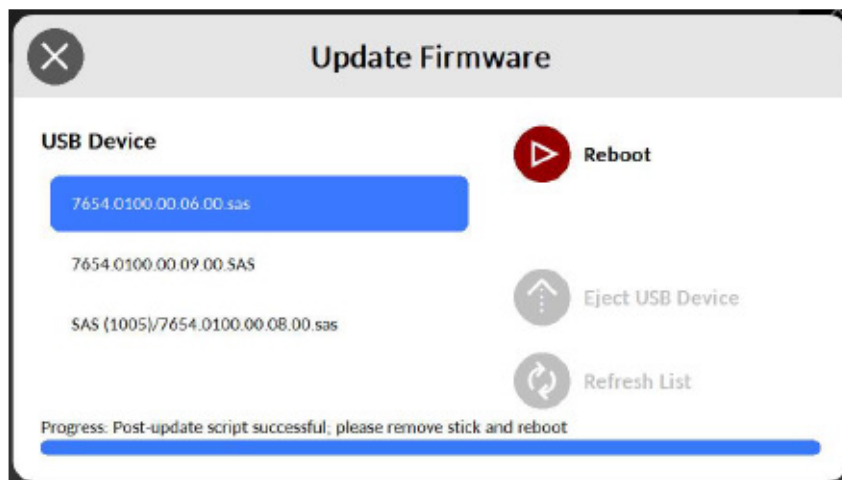

10. Check the update has been completed and that the Vitrési contains the new methods file/firmware version by going to **Maintenance > About**.

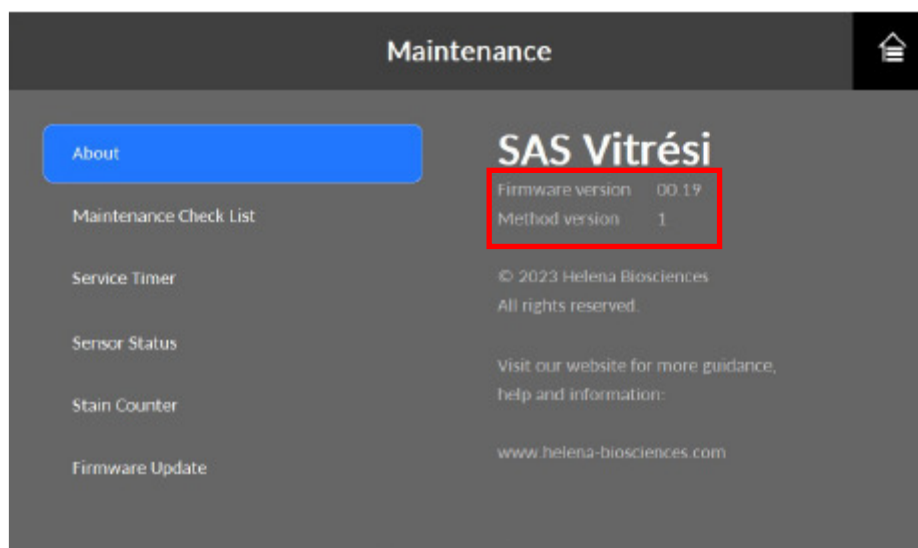

Please note: Firmware and methods files need to be updated independently.

Contact

For assistance and support, please contact Helena Biosciences Technical Support.